

TheraTogs Technical Highlight

Remember to:

- After laundering items, hang dry the garments with the foam facing outward. Avoid overlapping the white foam when it is wet.
- Refrain from pulling TheraTogs garments apart that were soiled, folded and placed in a diaper bag. Wash system in folded configuration before pulling apart.
- Following washing instructions to remove stains and dirt absorbed by the white foam surface.

When storing TheraTogs:

- Hanging the garments is the best option.
- If folding the garments, place white paper or towels on the foam side to protect the foam.

www.theratogs.com

Clinical Support questions: support@theratogs.com

Updated 10/2009

Preventing Damage to the White Foam Surface on TheraTogs.

Accidents happen. Knowing how to care for TheraTogs garments and taking precautions will prevent unwarranted damage to the foam surface of the TheraTogs garments while traveling or during everyday trips.

Here is a common

scenario: A child spits up or has a G-tube that leaks. The tank top front is quickly removed, folded and placed in a bag while a new tank top is donned. The soiled tank top dries and when taken out

of the bag and unfolded, the foam sticks and as a result the foam tears or peels.

When a system is soiled and cannot be rinsed out quickly in a sink, fold the system so that the foam side faces out or place the garment foam side on an old towel and fold. The towel will prevent the foam-on-foam soiled surface from sticking.

Friendly reminders to protect your child's TheraTogs garment.

DO NOT:

- Store or leave garments in a car (especially in summer heat)
- Store or leave garments near a heating device (heater, radiator)
- Wad system up after wear or after soiled before laundering
- Fold a soiled or damp TheraTog foam-on-foam

Suggestions for Proper TheraTogs Storage

Post Washing System:

- When hang drying please dry with foam side out, avoid foam overlapping or touching foam on foam when wet
- Store cleaned dry system on a hanger, foam side facing out
- Avoid using clips to hold system on hangers
- Protect foam from sharp items

Longer term storage:

- If the system is not to be worn, hang system in a closet avoiding foam on foam contact
- If folding system, place pieces of white paper or white tissue paper (avoid using colored papers, dyes will transfer) between white foam to protect the foam
- Minimize the amount of folding when storing long term

- Make sure all stains are removed and the system is properly and adequately washed before storing
- For disinfection, wash system with hydrogen peroxide
- For more effective washing, wash multiple times in a washing machine, rather than by hand

Store TheraTogs in a cool dry place.

Do not leave systems in a car, especially in the summer heat.