

TheraTogs™ ULTRA Posture & Torso Alignment (PTA) System

It's as **EASY** as 1 - 2 - 3



1 Fit the Garment

- ✓ The Hipster provides hip stability and attachment points for the TankTop and for strapping.
- ✓ The TankTop hugs the upper trunk.
- ✓ Together, the Hipster and TankTop create a full body 'field' for postural-assist strapping.

2 Align the Spine

- ✓ Choose from among 3 strapping applications to improve upper (shown) and/or lower trunk extension.
- ✓ If needed, strap to retract protracted shoulders and stabilize the scapulae.



3 Add Advanced Strapping

- ✓ Choose a strapping application to assist oblique and/or transverse abdominals to complete the core application.
- ✓ Strap from pelvis to thorax to reduce excess anterior pelvic tilt.



**TheraTogs ULTRA Systems
feature our new GoldTone™ fabric**

THERATOGS™
Therapy You Wear **ULTRA**



If your client has...	The PTA can provide...
Hypotonia	Shortening of trunk muscles - letting them work in, and adapt to, a shortened state
Flexible kyphosis	Improved posture and respiratory capacity
Flexible lordosis / excess pelvic tilt	Improved posture
Difficulty maintaining midline trunk and sitting balance; weak trunk muscles	Reduced deviations between thorax and pelvis; increased sensory awareness
Shoulder protraction & scapular abduction	Gentle shoulder retraction and scapular stabilization

NEW GoldTone™ fabric was developed during three years of R&D and field trials to meet the demanding performance requirements of TheraTogs systems. GoldTone is specifically formulated for the demands of biomechanical retraining, and manufactured using a patented fabric finishing process.

For Product & Application Guidance, contact Clinical Support:
support@theratogs.com | (888) 634-0495 ext. 307

Model Sizing & Selection Chart		Weight Lbs	Max Shoulder-to- Hip inches	Max Hip-to-Knee inches
Model	Model Name			
GPTA110	Posture & Torso Alignment System: Infant	15 – 25	10	9
GPTA210	Posture & Torso Alignment System: Pre-School	25 – 45	13	11
GPTA310	Posture & Torso Alignment System: Pediatric	45 – 65	15	18
GPTA410	Posture & Torso Alignment System: Petite Adult	65 – 110	17	20

How to Order

- ✓ TheraTogs are carried by leading distributors in the physical rehabilitation field all over the globe. See www.theratogs.com/buy for distributor names and contact links.
- ✓ (US only) TheraTogs may be ordered through your preferred orthotist or O&P clinic, particularly if reimbursement is a requirement. See www.theratogs.com/prolook for a list of Providers in your area.

About TheraTogs

TheraTogs™ are FDA Class I medical devices intended to be issued by, and applied under the supervision of, a licensed healthcare practitioner engaged in neuromotor or sensorimotor retraining. TheraTogs ULTRA systems are made of GoldTone™ - a proprietary and patented Latex®-free composite fabric with a unique foam backing made of an aqueous-based elastomeric urethane. TogRite™ strapping is Latex-free, elastomeric strapping with an inert, silicone-based grip surface.



TheraTogs are U.S. and foreign patent pending.

TheraTogs, Inc. • 305 Society Drive, Suite C-3 • Telluride, CO USA 81435
 (888) 634-0495 • (877) 202-5965 fax • www.theratogs.com • info@theratogs.com